*Tender Care*

ELECTRIC BREST PUMP

**SAVE YOUR TIME**

**MANUAL INSTRUCTION**

designed in EU

V1.2W

[www.lorelli.eu](http://www.lorelli.eu)

find us on

# CONTENT

<b>EN</b>	Manual Instruction.....	4
<b>BG</b>	Инструкция за употреба.....	10
<b>RO</b>	Instrucțiuni de utilizare.....	17
<b>FR</b>	Mode d'emploi.....	25
<b>IT</b>	Istruzione per l'uso.....	31
<b>GR</b>	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΚΑΡΕΚΛΑΣ.....	38
<b>ESP</b>	Instrucciones de uso.....	45
<b>RU</b>	Инструкция по эксплуатации.....	51
<b>SRB HR</b>	Uputstvo za upotrebu.....	59
<b>ME BIH</b>		
<b>HU</b>	Használati utasítás.....	65
<b>AL</b>	Instrukcion për përdorim.....	71
<b>NL</b>	Gebruikshandleiding.....	77
<b>DE</b>	Gebrauchsanweisung.....	83
<b>PL</b>	Instrukcja użytkownika.....	89
<b>MK</b>	Упатствата за употреба.....	95
<b>CZ</b>	Návod k použití.....	102
<b>TR</b>	Kullanım talimatı.....	109
<b>AR</b>	دليل التعليمات.....	117

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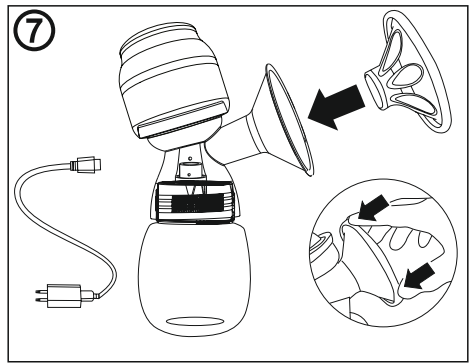
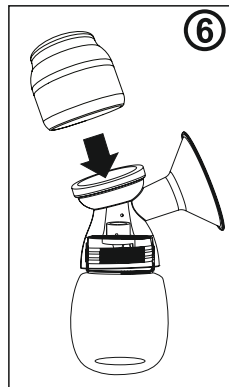
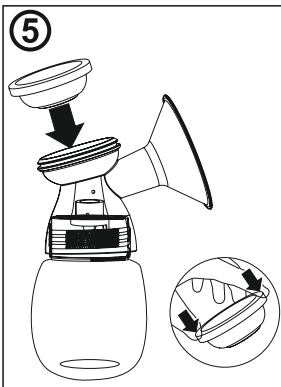
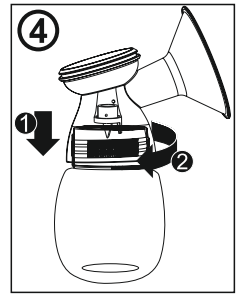
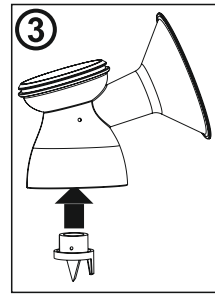
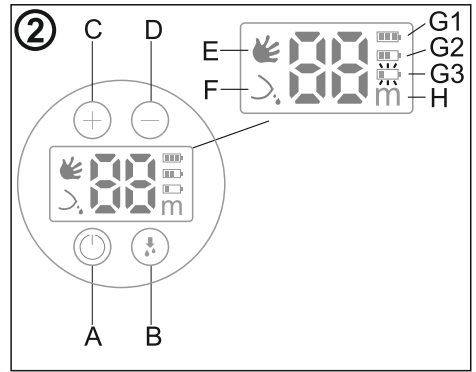
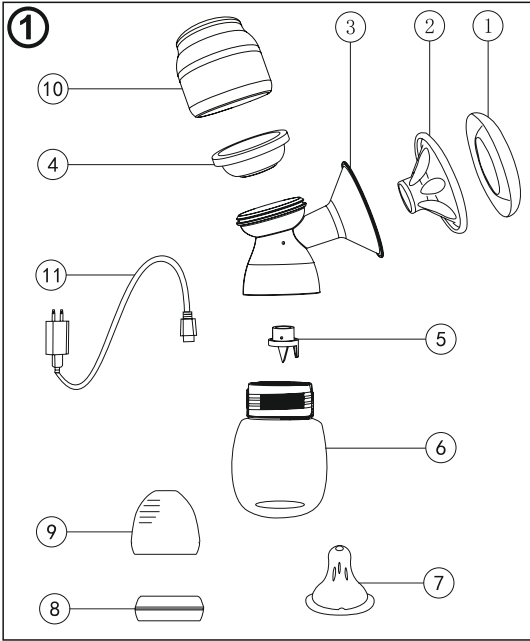
**Electric breast pump SAVE YOUR TIME****IMPORTANT SAFETY INFORMATION – READ THIS USER MANUAL CAREFULLY BEFORE YOU USE THE BREAST PUMP AND SAVE IT FOR FUTURE REFERENCE.****Intended use**

The LORELLI electric breast pump is intended to express and collect milk from the breast of a lactating woman. The device is intended for a single user.

**SAFETY REQUIREMENT****Contraindications**

**WARNING!** Never use the breast pump while you are pregnant, as pumping can induce labor.

To prevent injury and damage, important notes are explained using various symbols. The symbols and their meanings are as follows:

**WARNING** - Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injuries.

**CAUTION** - Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injuries.

**WARNING****Warnings to avoid choking, strangulation and injury:**

- Do not allow children or pets to play with the motor unit, the adapter or accessories.
- Children should be supervised to ensure that they do not play with the appliance.
- Always unplug the breast pump immediately after use.
- Do not leave the breast pump unattended when it is plugged in.
- This breast pump is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised by or have received proper instruction concerning use of the appliance by a person responsible for their safety.

**Warnings to avoid electric shock and burns:**

- Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug is damaged, if it does not work properly or if it has been dropped or submerged in water.
- Use only the specified voltage and the suggested AC adapter to prevent overheating of the adapter, any damage or failure.
- Do not touch the AC adapter and the breast pump during a thunder storm.
- Do not place the breast pump/ battery/AC adapter in direct sunlight or rain, near heaters or other hot appliances, in any location exposed to extremely high or low temperatures, high humidity or in dusty locations.
- Do not place heavy objects on, or damage the power cord or plug. Do not forcibly bend or pull the power cord. Always hold the AC adapter when disconnecting from the power outlet. Make sure that the plug is fully inserted to the power outlet. Do not use an outlet that is loose.
- Do not allow the breast pump / AC adapter/ power plug/ battery to get wet.
- Do not disassemble or modify the breast pump/AC adapter/ battery. Do not throw the battery into fire or expose to heat.
- Disconnect the AC adapter and remove battery immediately and stop using the breast pump if you notice abnormal odor, heat, discoloration, deformation or anything unusual while using or storing it.

### **Warnings to avoid poisoning and contamination and to ensure hygiene:**

- For hygienic reasons, the breast pump is only intended for repeated use by a single user.
- Clean, rinse and sanitize all pump parts, except the motor unit and the adapter, before each use.
- Only store breast milk collected with a cleaned and sanitized pump.
- Do not use the breast pump if the silicone diaphragm appears to be damaged or broken.
- Do not use antibacterial or abrasive cleaning agents when you clean breast pump parts.

### **Warnings to avoid breast and nipple problems and pain:**

- Do not attempt to remove the pump body from your breast while under vacuum. Switch off the appliance and break the seal between your breast and the pump funnel with your finger. Remove the pump from your breast.
- Never use the breast pump while you are sleepy or drowsy to avoid lack of attention during use.
- Always switch off the breast pump before you remove the pump body from your breast to release the vacuum.
- Never use any accessories or parts from other manufacturers or any accessories or parts which are not specifically recommended from the manufacturer. This could lead to improper working of the appliance and could influence Electromagnetic Compatibility (EMC). If you use such accessories or parts, your warranty becomes invalid.
- Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.
- If the process becomes very uncomfortable or painful, stop using the pump and consult your breast feeding advisor.
- If the pressure generated is uncomfortable or causes pain, switch off the appliance, break the seal between the breast and the pump body with your finger and remove the pump from your breast.
- Never drop or insert any foreign object into any opening.

## **CAUTION**

### **Caution to prevent personal injury, damage to and malfunctioning of the product:**

- Prevent the adapter and motor unit from coming into contact with water.
- Keep the adapter and the motor unit away from heated surfaces to avoid overheating and deformation of these parts.
- Never put the motor unit or the adapter in water, in a dishwasher or in a sanitizer.
- Although the breast pump is compliant with applicable EMC directives, it may still be susceptible to excessive emissions and/or may interfere with other equipment. A consequence can be that the breast pump switches off or goes into error mode. To prevent interference, keep other electric equipment away from the breast pump during expressing (see 'Technical information').
- There are no user serviceable parts inside the motor unit of the breast pump. No modification of equipment is allowed. If you do, your warranty becomes invalid.
- Do not drop or hit the breast pump / AC adapter. Do not press on the LCD display.
- The motor unit and the adapter do not require cleaning and sterilizing, as they do not come into contact with breast milk.
- Never use alcohol or other organic solvents to clean the breast pump.
- Do not use the breast pump in any way or for any purpose not described in this guide. Doing so may result in accidents or damage the breast pump.

## **PARTS**

### **General description (Picture 1)**

1. Cover - 1 pcs. ;
2. Massage cushion - 1 pcs. ;
3. Pump body - 1 pcs. ;
4. Silicone diaphragm- 1 pcs. ;
5. Valve - 1 pcs. ;
6. Bottle - 1 pcs. ;
7. Teat - 1 pcs. ;
8. Ring - 1 pcs. ;
9. Cap - 1 pcs. ;
10. Motor unit - 1 pcs. ;
11. Adapter - 1 pcs. ;

## PREPARING FOR USE

### Cleaning and sanitizing

The motor unit and the adapter do not require cleaning and sterilizing, as they do not come into contact with breast milk. Clean and sanitize all other parts of the breast pump before first use. Also clean these parts after each subsequent use and sanitize them before each subsequent use.

**Caution!** : Never put the motor unit or the adapter in water, in a dishwasher or in a sanitizer, as this causes permanent damage to these parts.

1. Disassemble the breast pump completely. Also remove the valve. Be careful when you remove the valve and when you clean it. If it gets damaged, your breast pump does not function properly. To remove the valve, gently pull at the ribbed tab on the side of the valve.

2. Clean all parts, except the motor unit, and the adapter, in the dishwasher (on the top rack only) or in hot water with some mild washing-up

liquid and then rinse them thoroughly. To clean the valve, rub it gently between your fingers in warm water with some washing-up liquid. Do not insert objects into the valve, as this may cause damage.

**Warning!** - Do not use antibacterial or abrasive cleaning agents when you are cleaning.

**Note:** Make sure you have cleaned and sanitized the appropriate parts of the breast pump.

Wash your hands thoroughly before you handle the cleaned parts.

**Warning!** - Be careful, the cleaned parts may still be hot. Only start assembling the breast pump when cleaned parts have cooled down.

### LCD DISPLAY & KEY FUNCTION ( Picture 2 )

A. On-Off Key ; B. Mode Key ; C. Increase Vacuum ; D. Reduce Vacuum E - Massage mode;

F- Pumping mode; G1 - Fully charged battery; G2 - Half charged battery;

G3 - Blinking signal - low battery; H - Time of work ;

### BATTERY OPERATION ( Picture 2 )

**WARNING!!! Before first use, fully charge the battery for 8 hours uninterrupted!** Once the pump is started, the digits on the display will show the remaining battery life until the battery runs out, and then the selected vacuum level will be displayed.

**G-1;2;3;** When the battery indicator appears on the display, and the indicator (G1) on the battery stop blinking, the battery is fully charged.

G-1 Fully charged battery ; G-2 - Half charged battery ; G3- Blinking signal - Low battery

**Attention: Battery can not be charged over 24 hours.**

## FEEDING BOTTLE 180ml

### Cleaning and sanitizing FEEDING BOTTLE 180ml

- Before first use, disassemble and clean the product and then put the components in boiling water for 5 minutes. This ensures hygiene.
- During disinfection with boiling water, do not allow the bottle or other parts to touch the side of the pot. This may cause irreparable deformation or damage to the product for which Lorelli is not responsible.
- Clean carefully before each subsequent use to ensure hygiene.
- Wash all parts with warm water and delicate detergent. Always rinse all parts thoroughly after each wash.
- Wash thoroughly and rinse all parts with clean water, including areas that may not be visible but through which the food has passed, to remove food debris. Make sure there is no liquid left in the pacifier.
- Do not place in a microwave oven. Do not heat the product in traditional ovens, on hot plates or open flames.
- Avoid the use of sharp or abrasive materials, as this may damage the product.
- Do not clean or allow the product to come into contact with solvents or aggressive chemicals. This may damage the product.
- If you use bottle brushes, be careful not to damage the most delicate parts of the product.
- After cleaning the product, dry all parts thoroughly and store in a clean and dry place without direct sunlight and away from heat sources.
- Do not leave the pacifier in direct sunlight or heat or leave it in a disinfectant ("sterilizing solution") for longer than recommended, as this may damage the pacifier. Note: Be sure to clean and disinfect the relevant parts of the breast pump. Wash your hands thoroughly before working on clean parts!

## BABY FEEDING

- Inspect the feeding teat before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness.
- Never dip the teat in sweet substances or medication, your child may get tooth decay.
- Do not microwave. Do not heat the product in traditional ovens or on hot plates or open flames.

## FOR YOUR CHILD SAFETY AND HEALTH

### WARNING!

- Always use this product with adult supervision. Accidents have occurred when babies have been left alone with drinking equipment due to the baby falling or if the product has disassembled.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay. Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the bottle for long periods through the day and particularly through the night, when saliva flow is reduced or if it is used as a soother.
- Always check food temperature before feeding.
- Keep components not in use out of the reach of children.
- Throw away at the first signs of damage or weakness.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.

## OPERATION

### Display and buttons ( Photo 2 )

**A.** ON/OFF button; **B.** Mode button; **C.** Increasing the vacuum; **D.** Vacuum reduction; **E** - Massage mode; **F** - Squeeze mode; **G1** - Fully charged battery.; **G2** - Average battery charge; **G3** - Flashing signal - low battery; **H** - Working time;

### WORKING WITH THE BATTERY ( Picture 2 )

**ATTENTION!!! Before first use, fully charge the battery for 8 hours without interruption!** The remaining battery life indicator shows the time remaining until the battery is depleted. **G-1;2;3** When the screen shows full battery and the battery indicator (G1) stops flashing, the battery is fully charged.

**G1** - Fully charged battery.;

**G2** - Average battery charge;

**G3** - Flashing signal - low battery;

**Important! The battery cannot be charged for more than 24 hours.**

### ASSEMBLY OF THE BREAST PUMP

1. Insert the valve into the pump body from underneath. Push the valve in as far as possible. (Picture 3 )
  2. Screw the pump body clockwise onto the bottle until it is securely fixed. (Picture 4)
  3. Insert the silicone diaphragm into the pump body from above. Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal. (Picture 5 )
  4. Place the motor unit over the silicone diaphragm and push until “ Click” is heard. Ensure that the motor unit is firmly fixed to the pump body!
  5. Insert the massage cushion into the funnel-shaped section of the pump body. Push in the inner part of the cushion as far as possible and make sure that it is perfectly sealed all around the rim of the pump body. Press in between the petals to remove any trapped air. **Warning!** Always use the breast pump with a cushion.
- Insert the adapter into the wall socket and insert the plug at the other end into the motor unit. (Picture 7).

## SETTINGS

Press the key On/Off – Massage mode automatic starts.

Press the mode key - it switches Massage, Pumping mode, Deep pumping mode.

▲ Press + or - key , LCD screen shows the relevant level.

▲ The default pumping time of the pump is 30 minutes, the pump will automatic power off after the time is up. It will memory your typical manner of pumping before powering off.

▲ The milk level in the bottle can't exceed the valve level!

▲ Press and hold the mode key for a while to select the mixed mode:massage+pumping mode /massage + Deep pumping mode.

● You can adjust the suction level with the buttons „+“ and „-“

● Massage mode - 10 levels

Pumping mode , Deep pumping mode - 10 levels (0-9)

## WHEN TO EXPRESS MILK

If breastfeeding goes well, it is advisable (unless advised otherwise by your healthcare professional/ breastfeeding advisor) to wait until your milk supply and breastfeeding schedule are established (normally at least 2 to 4 weeks after giving birth) before you start expressing.

### Exceptions:

— If breastfeeding does not go well immediately, expressing regularly can help to establish and maintain your milk supply.

— If you are expressing milk for your baby to be given in hospital.

— If your breasts are engorged (painful or swollen), you can express a small amount of milk before or between feeds to ease the pain and to help your baby latch on more easily.

— If you have sore or cracked nipples, you may wish to express your milk until they are healed.

— If you are separated from your baby and wish to continue to breastfeed when you are reunited, you should express your milk regularly to stimulate your milk supply.

You need to find the optimum times of the day to express your milk, for example just before or after your baby's first feed in the morning when your breasts are full, or after a feed if your baby has not emptied both breasts. If you have returned to work, you may need to express during a break.

Using a breast pump requires practice and it may take several attempts before you succeed.

Fortunately, the Lorelli electric breast pump is easy to assemble and use so you will soon get used to expressing with it.

## OPERATING THE BREAST PUMP

1. Wash your hands thoroughly and make sure your breasts are clean. Relax in a comfortable chair.

2. Press the assembled pump body against your breast. Make sure that your nipple is centered, so that the massage cushion creates an airtight seal. Make sure that your nipple fits correctly in the cushion.

3. Press the on/off button. The breast pump automatically starts up in massage mode and you can change the level of the massage through the increase & reduce keys.

4. When the breast milk flows, you can change the mode from massage to normal pumping (through Mode key) and choose the most comfortable level.

Attention: Do not always use the maximum level of pumping, just choose the most comfortable level for pumping.

Note: Do not worry if your milk does not flow immediately. Relax and continue pumping. The first few times you use the breast pump, you may need to use a higher suction mode to get your milk flowing.

5. Depending on your own personal comfort, you may wish to use a higher suction mode that provides deeper suction. You can always return to a lower suction mode.

6. When you have finished expressing, turn off the breast pump with the ON/OFF key and carefully remove the pump body from your breast.

7. Unscrew the bottle from the pump body. Feed your baby or store the milk.

8. Unplug the adapter from the wall outlet to disconnect the appliance from the electrical power.

Unplug the motor unit from the adapter. Unplug the motor unit and the silicone diaphragm.

9. Clean the other used parts of the breast pump according to the instructions in section

'Cleaning and sanitizing'.

## STORING BREAST MILK

**Warning!** Only store breast milk collected with a cleaned and sanitized pump to ensure hygiene.

— Breast milk can be stored in the fridge (not in the door) for up to 48 hours. Expressed milk should be refrigerated immediately. If you store milk in the fridge to add to it during the day, only add milk that has been expressed into a sanitized bottle.

— Breast milk can be stored in the freezer for up to three months as long as it is kept in sanitized bottles fitted with a sanitized screw ring and sealing disc. Clearly label the bottle with the date and time of expression and use older breast milk first.

— If you intend to feed your baby with the expressed breast milk within 48 hours, you can store the breast milk in the fridge in an assembled bottle.

### **WARNING!**

Always refrigerate or freeze expressed milk immediately.

Only store milk collected with a sanitized breast pump in sanitized bottles.

Never refreeze thawed breast milk.

Never add fresh breast milk to frozen breast milk.

## TROUBLESHOOTING

**Problem:** I experience pain when I use the breast pump.

**Solution:** One of the reasons can be the too high vacuum level. Maybe you don't need the full vacuum power of the breast pump. At first use the Massage mode as a massage stimulation and after that you can use the express mode. If you still feel a pain, stop using the breast pump and consult your breastfeeding advisor. If the pump does not come away easily from the breast, it may be necessary to break the vacuum by placing a finger between the breast and Massage Cushion.

**Problem:** The breast pump does not work.

**Solution:** First check if you have assembled the breast pump correctly. To prevent interference keep other electric equipment, like mobile phone or laptop, away from the breast pump while expressing. Check the battery or if you are using the breast pump on electrical power, check to make sure you are using the adapter that was supplied with the product.

**Problem:** I do not feel any suction.

**Solution:** Check if you have assembled the breast pump correctly by following the steps in section 'preparing for use'. Make sure that the valve and silicone diaphragm are correctly placed and the motor unit is firmly attached over the silicone diaphragm and to the body of the pump. Also make sure that the pump is positioned correctly on the breast to allow for vacuum to occur.

**Problem:** The pump has too much suction.

**Solution:** Follow below guidelines if you experience too much suction with your breast pump:

Make sure you only use the parts recommended from the manufacturer.

Make sure you have assembled the breast pump with the massage cushion: not using the massage cushion can result in too much vacuum.

When using a breast pump for the first time, you may experience the suction level as too high at first. Practicing can help. If the problem persists, consult your breast feeding advisor.

**Problem:** I encounter back flow issue during pumping.

**Solution:** Immediately shut off the breast pump, remove the bottle with the milk and motor unit. Check for milk in the valve and diaphragm, clean and dry. If the stern has not reached the drive unit and the diaphragm is strong, you can continue to use the pump. If the diaphragm is damaged, you need to replace it. Please contact the manufacturer if breast milk has enter the motor & do not continue to use it.

## USAGE AND STORAGE CONDITIONS

Keep the breast pump out of direct sunlight as prolonged exposure may cause discoloration.

Store the breast pump and its accessories in a safe, clean and dry place.

If the appliance has been stored in a hot or cold environment, place it in the usage environment to let it reach a temperature within the usage conditions ( 10°C to 30°C ) before you use it.

## TECHNICAL SPECIFICATIONS

Sucking power: 60mmhg--380mmhg

Noise limited: <60dB

Fixed time : 30 minutes

Lithium battery 3.7V 1100mAh

IP Class: IPX0

Environmental Conditions for Operation : Temperature 10-30°C, Relative humidity 15%-93%, Atmospheric pressure 860-1060hpa

Environmental Conditions Storage : Temperature -25-50°C , Relative humidity 15%-93% , Atmospheric pressure 860-1060hPa

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## DECLARATION OF CONFORMITY

Name and address of the manufacturer:

NINGBO YOUHA ELECTRICAL TECHNOLOGY CO., LTD  
(NINGBO YOUHE MOTHER AND BABY PRODUCTS CO.,LTD)  
567# Xintang Road, Fuhai, Cixi, Zhejiang, China

We hereby declare that our product, item number YH-8103 and the components of our product delivered to DIDIS Ltd as 1022060 ELECTRIC BREAST PUMP SAVE YOUR TIME comply with the requirements of the below listed directive and European standards:

1. DIRECTIVE 2014/30/EU OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 26 February 2014 (EMC)

The product conforms to the following standards:

EN IEC 55014-1

EN IEC 61000-3-2

EN 61000-3-3

EN IEC 55014-2

2. DIRECTIVE 2014/35/EU OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 26 February 2014 (LVD)

The product conforms to the following standards:

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NINGBO YOUHA ELECTRICAL TECHNOLOGY CO., LTD  
(NINGBO YOUHE MOTHER AND BABY PRODUCTS CO., LTD)  
567# Xintang Road, Fuhai, Cixi, Zhejiang, China

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MOTHER&BABY PRODUCTS CO.,LTD.
Importator:Didis Ltd.
PRODUCTS CO.,LTD. Zhejiang, China